




NOVAKM[®]
bring treatment to a higher level

Day clinic chairs

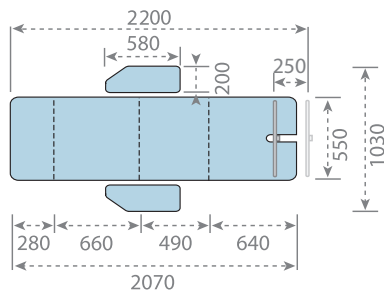

THERAPY CHAIR




ZeroG exclusive



Model: TCZ-EX

Make the **TIME FLY** in **perfect comfort** during lengthy therapies.


Therapy chairs ZeroG are distinguished by the **soft mattress** that prevents sweating and overheating, **adjustable head section** and **armrests** and the **ZERO GRAVITY POSITION** that relieves the pressure of gravity from the body by distributing the body's weight evenly.

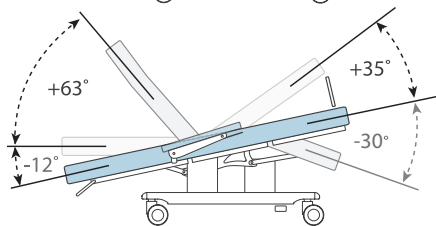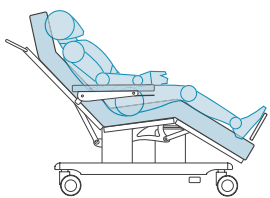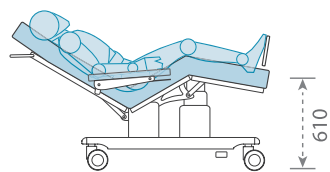

POSITIONS



CONFIGURE

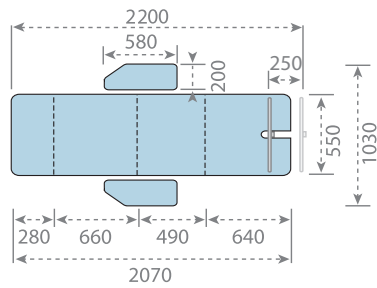

THERAPY CHAIR

ZeroG luxury






Model: TCZ-LUX

»The hand switch is easy to operate and the illustrations are clear and understandable. When I put the chair in ZeroG position, I didn't move until the end of the therapy. Due to the exceptionally soft and comfortable upholstery, I was calm and without back pains. It felt like I was sitting on a soft cloud.«

(Maja, patient at dialysis department)

»Therapy chair ZeroG is a luxury for the patients, as they can set it into desirable position all on their own. Height adjustment is luxurious especially for the medical staff, because we can take better care of our back.«

(Mojca, nurse at rheumatology department)

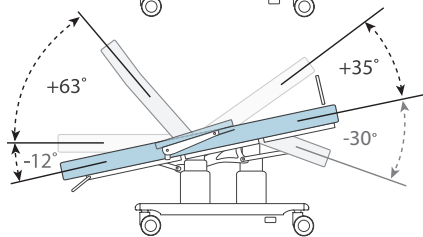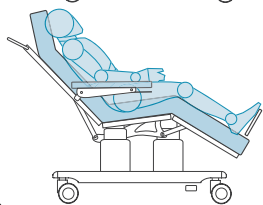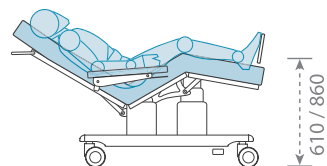

POSITIONS



Equipment and accessories

standard ● optional ○

	TCZ-EX	TCZ-LUX
height of the chair - upholstery included	61 cm	61-86 cm
height of the chair - upholstery excluded	47 cm	47-72 cm
safe working load	250 kg	250 kg
push handles	●	●
paper roll holder – NZP1-Z	○	○
adjustable footrest – FR-Z	○	○
cushion for footrest – BZN1-Z	○	○
infusion stand holder, left – SIFH1-Z-L	○	○
infusion stand holder, right – SIFH1-Z-R	○	○
infusion stand – SIF1-Z	○	○
fixation rail at sitting section (pair) – LE1-Z	○	○
armrest side protection (pair) – HRP-Z	○	○
PVC cover for leg section – PVC1-Z	○	○
PVC cover for leg section with footrest – PVC1-FR-Z	○	○
PVC cover for armrest – PVC3-Z	○	○
castors with central locking system (Ø 100 mm) – SER-CK1-Z	●	●
antistatic castors with central locking system (Ø 100 mm) – AC1-CK-Z	○	○
large antistatic castors with central locking system (Ø 125 mm) – AC2-CK-Z	○	○
padding for heavy use - dark sky blue – HPU-Z	○	○
plug connector for potential equalization – PPEQ	○	○
battery backup – BB-Z	○	○

Color chart PU

- 
231 light sky gray
standard
- 
132 sky blue
standard
- 
133 dark sky blue
standard or padding for heavy use (HPU-Z) at extra-cost



Comfort


The **soft padding** and setting the therapy chair into **Zero Gravity position** guarantee first-class comfort for patients during long-lasting therapies. The upholstery is made of **high-quality foam** that provides **full body support** along the lying surface and prevents **overheating** and **sweating**. By pressing the **“cloud” button** on the hand switch, the patient can set the chair into **ZeroG** position. This offers additional **pressure relief for the body and joints** during longer therapies.

Easy adjustment

A wide range of smart and simple features save handling time for medical staff, as patients can **adjust the chair independently** into their favourable position. By pressing a single button on a hand switch, the patient can set the chair into **ZeroG position, sitting position** or **completely flat position**. In case of emergency you can quickly set the chair into the **Trendelenburg** position. All chair sections can be adjusted **simultaneously**. For even greater comfort, the patient can easily adjust the **armrests** by pressing the lever and the **head section**. The chair can be additionally equipped with a **footrest** that supports patients in case of cramps. By pressing and releasing the built-in button they can quickly adjust the footrest themselves.

Simple cleaning and maintenance

Therapy chair ZeroG has a **small footprint** so it takes very little space in a clinic. Simply designed lying surface is smooth and without folds for easier and faster cleaning. It is made of thin, **extra strong PU material**, which is resistant to blood, urine, oil, fat and disinfectants to **ensure proper hygiene** for your patients.

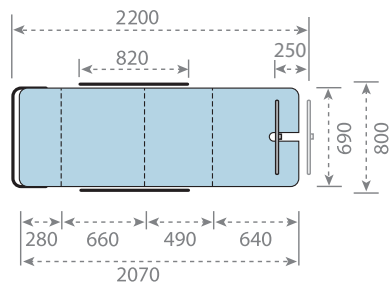

DAY HOSPITAL CHAIR ZeroG






Model: DHCZ


The **Day hospital chair ZeroG** was developed in response to the latest trends in **outpatient surgery** and **postoperative recovery**.

With a design that enables **easy** and **safe transport** and **facilitates extremely comfortable recovery** after surgery, the new ZeroG chair perfectly replaces both a standard **PATIENT TRANSPORT TROLLEY** and an electric **HOSPITAL BED**.

The chair was designed to **prepare patients** for surgery and make their **transport** and **rest** after surgery **safe and comfortable**.

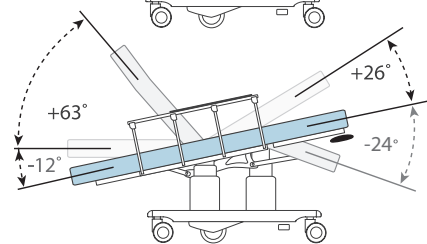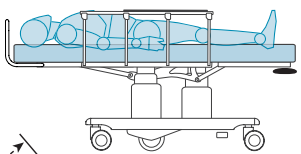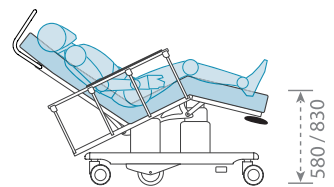

POSITIONS

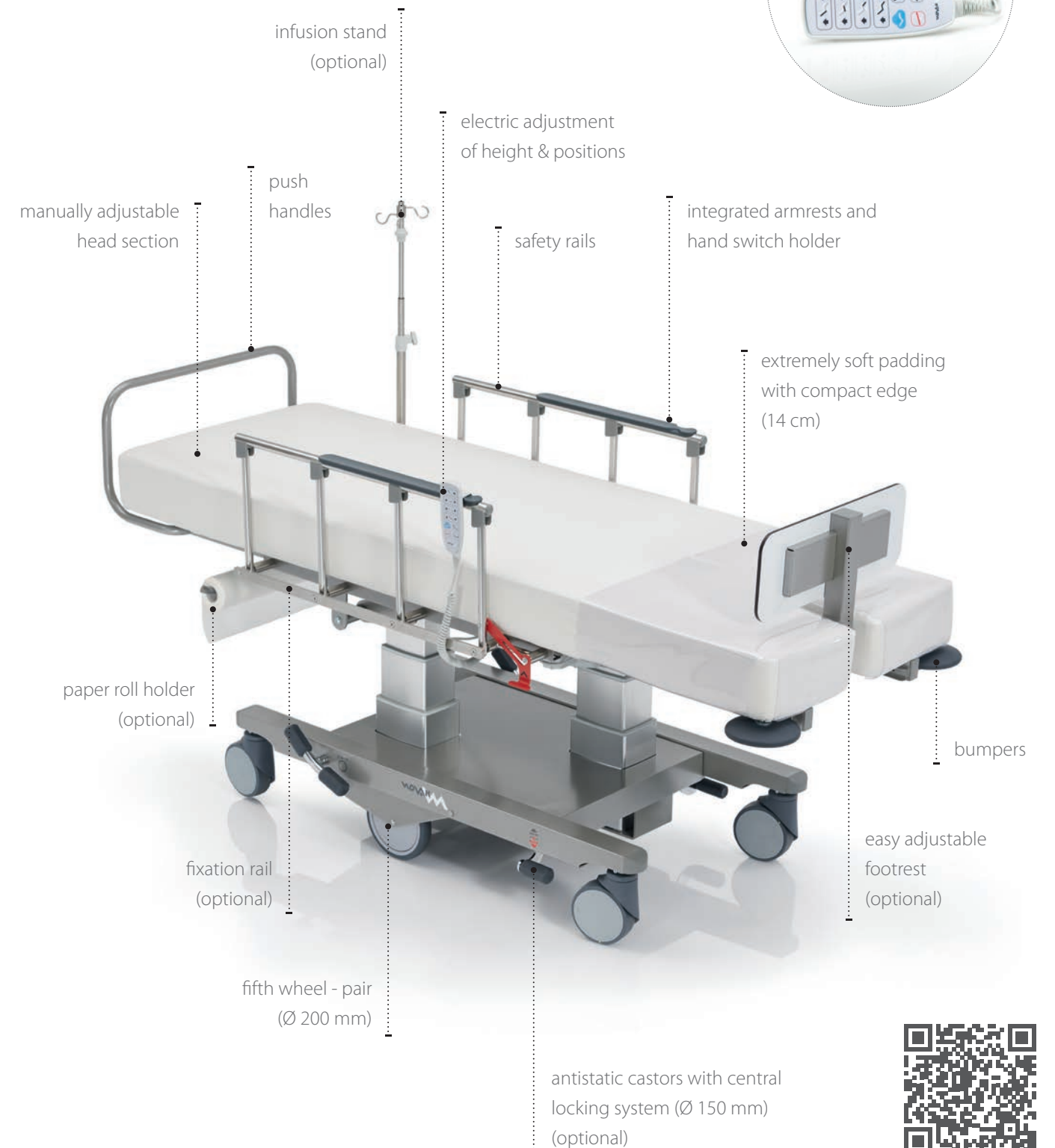

Equipment and accessories

standard ● optional ○

	DHCZ
height of the chair - upholstery included	58 - 83 cm
height of the chair - upholstery excluded	44 - 69 cm
safe working load	250 kg
push handles	●
bumpers	●
safety rails	●
integrated armrest and hand switch holder	●
paper roll holder – NZP2-Z	○
adjustable footrest – FR-Z	○
infusion stand holder, right – SIFH2-Z-R	○
infusion stand holder, left – SIFH2-Z-L	○
infusion stand – SIF1-Z	○
fixation rail at sitting section (pair) – LE2-Z	○
black antistatic padding – APU-Z	○
padding for heavy use - dark sky blue – HPU-Z	○
PVC cover for leg section – PVC2-Z	○
PVC cover for leg section with footrest – PVC2-FR-Z	○
fifth wheel (pair) (Ø 200 mm)	●
castors with central locking system (Ø 150 mm) – SER-CK2-Z	●
antistatic castors with central locking system (Ø 150 mm) (4 pcs) – AC3-CK-Z	○
battery backup – BB-Z	○
plug connector for potential equalization – PPEQ	○
oxygen bottle holder (45 cm) – O2-Z	○
extended safety rails – OOLE-Z	○

Color chart PU

- 
231 light sky gray
standard
- 
132 sky blue
standard
- 
133 dark sky blue
standard or padding for heavy use (HPU-Z)
- 
PU black
black, antistatic padding (APU-Z)



Easy access and preparation

The height of the Day hospital chair ZeroG is electrically adjustable by a hand switch. One of the chair's key advantages is the **low sitting position** that enables patients' easier access on the chair. **Integrated armrests** on **safety rails** (patent EP Nr. 3496694 and US Nr. 11110019) allow the medical staff to prepare the patient for surgery.

The chair can be set into a **completely flat position**, so it is perfect for all patients' transfers from the day hospital chair ZeroG to any OR table and vice versa. An **infusion stand**, which can be attached either on the left or on the right side of the chair, is always placed **vertically**, regardless of the chair's position.

Exceptional mobility and agility

A wide range of smart and simple features enable easier pushing, turning and driving. The day hospital chair ZeroG has **castors with central locking system**, which are adjustable from both sides of the chair. To provide easier driving along corridors, excellent manageability and turning on the spot, the chair is additionally equipped with **two fifth wheels**.

Push handles on optimal height at head section provide a safe grip during driving, while **bumpers** on the corners at leg section protect the chair and surrounding area from damage.

Comfortable and safe recovery

Simple adjustments of the Day hospital chair ZeroG allow surgeons to safely transfer the patients onto the chair after surgery and place them into an **optimal position for recovery**. Firm and solid **safety rails** on both sides of the ZeroG chair protect patients against falls.

The **extremely soft padding** and **simple adjustments** guarantee first-class comfort for patients during recovery after surgery. The patients can adjust the ZeroG chair **independently** into their favourable position. By pressing a single **"cloud" button** on a **hand switch**, they can set the chair into **ZeroG position**, the most comfortable position for resting. Anytime during recovery, the hand switch can be securely placed on a **hand switch holder** that is integrated along with armrests on safety rails.



.....

Novak M d.o.o.

Pod kostanji 7
SI - 1218 Komenda

E: info@novak-m.si
T: +386 1 5620117

www.novak-m.com

